



## **INSTRUCTIONS FOR USE**

### **MUMU 3-PLY SURGICAL MASK**

#### **(FACIAL MASK FOR MEDICAL USE)**

##### **Intended Use:**

3-ply surgical masks (facial mask for medical use) is intended to be worn by medical personnel during surgical or other medical procedures to protect both the patient and the operating personnel and any other person that want and need to be protected, from transfer of microorganisms, body fluid, particulate material transfer and any other microbes.

Reduces exposure to blood and body fluids. Minimizes contamination to exhaled microorganisms. This product is intended for use in infection control practices.

##### **Technical Specifications:**

Non-irritating, Fluid Resistant, Three Ply construction.

3 pleats of folds to allow the user to expand the mask so it covers the area from the nose to the chin. Mask is secured with an ear loop to be placed behind the ears.

The surgical mask's (facial mask for medical use) 3-ply layers work as follows:

- **The outer layer** repels water, blood, and other body fluids.
- **The middle layer** filters certain pathogens.
- **The inner layer** absorbs moisture and sweat from exhaled air.

##### **Donning The Mask:**

- Before putting on the mask, wash your hands for at least 20 seconds with soap and water, or rub your hands together thoroughly with alcohol-based hand sanitizer.
- Check for defects in the face mask, such as tears or broken loops.
- Position the colored side of the mask outward.
- If present, make sure the metallic strip is at the top of the mask and positioned against the bridge of your nose.
- Ear loops: Hold the mask by both ear loops and place one loop over each ear.
- Mold the bendable metallic upper strip to the shape of your nose by pinching and pressing down on it with your fingers.
- Pull the bottom of the mask over your mouth and chin.



- Be sure the mask fits snugly.
- Don't touch the mask once in position.
- If the mask gets soiled or damp, replace it with a new one.

**!! Do not:**

- touch the mask once it's secured on your face, as it might have pathogens on it
- dangle the mask from one ear
- hang the mask around your neck
- reuse single-use masks.

If you have to touch the face mask while you're wearing it, wash your hands first. Be sure to also wash your hands afterward, or use hand sanitizer.

**Doffing The Mask:**

- Before you take off the mask, wash your hands well or use hand sanitizer.
- Avoid touching the mask itself, as it could be contaminated. Hold it by the loops, ties, or bands only.
- Carefully remove the mask from your face once you: unhook both ear loops.
- Holding the mask loops discard the mask by placing it in a covered trash bin.
- After removing the mask, wash your hands thoroughly or use hand sanitizer.

**STORAGE**

Capable of being stored continuously in ambient temperature of 10 to 35 deg C and relative humidity of 15 to 55%. Capable of operating continuously in ambient temperature of 10 to 35 deg C and relative humidity of 15 to 55%.

It is recommended that 3-ply surgical masks (facial mask for medical use) should be stored in their original containers and should be stored away from direct sunlight, heat sources and liquids, including chemicals. The area should be clean and should protect the masks from contamination. Never store it in a purse or pocket.